respimar

A 100% natural solution

Respimar is a range of seawater-based products for nasal hygiene that helps eliminate excess mucus in cases of nasal congestion.

It is a 100% natural solution with **plant extracts (aloe vera and camomile)** that preserves all the sea water minerals and trace elements beneficial for the proper functioning of the nostrils. Contains no propellant gas, preservatives or chemicals.

4 solutions depending on your needs

The water in Respimar comes from the Bay of Cancale on the French Brittany coast. The location is famous for its strong tides and open seas, which provide excellent quality sea water.

	respimar pediatric	respimar CLEANING AND HYDRATION	respimar DECONGESTANT	respimar DECONGESTANT GO
Age				
Concentration	Isotonic Concentration of salts the same as in human body cells (0.9%)		Hypertonic High concentration of ions (2.5%) to promote the: elimination of excess mucus	
Composition	Sea water and purified water with natural extracts of Aloe Vera and Camomile		Sea water and purified water. Extracts of Camomile, Aloe Vera, Mint and Propylene Glycol	
Diffusion type	 Smooth Ergonomic diffuser	 Medium Ergonomic diffuser	 Strong Ergonomic diffuser	 Smooth Nasal pump spray
Format	120 ml For home	120 ml For home	120 ml For home	30 ml On the go
Children and infants from 1 st day				
Children over 6 and adults				
Pregnancy				

Respimar: Nasal Hygiene and Decongestion

The nose is the gateway to the upper respiratory tract. The nasal mucosa (inner part of the nose) filters the air breathed in to remove dust, germs and any irritating particles it may contain.

A NASAL WASH is a hygiene measure to be performed daily, not only to eliminate nasal discharge, but also to release contaminating particles, viruses, bacteria and other impurities from the nasal cavities that may lead to increased respiratory complications.

If mucus is very abundant, it is difficult to remove or if it obstructs the passage of air; this is known as NASAL CONGESTION. It is the feeling of difficulty felt when breathing through the nostrils, caused by inflammation of the mucosa and the production of excess mucus as a defence mechanism against infections (e.g. colds, flu or rhinitis), inhaled irritants (e.g. environmental pollution or tobacco smoke) or allergic situations (e.g. pollen or mites).

The excess mucus and sensation of nasal obstruction is a very annoying symptom that can lead to disturbances in your daily routine and quality of sleep.

To alleviate nasal obstruction naturally, pressurised hypertonic solutions can be used to remove excess mucus.

Indications and posology

Being a 100% natural solution, it contains no drugs or chemicals which would make its use inadvisable after a certain period of time.

respimar
pediatric

respimar
CLEANING AND
HYDRATION

Indications

This sea water solution has the same concentration of salts as the nasal mucosa cells. The formula is designed for pathological processes accompanied by nasal secretion, with increased moisturisation due to the **Aloe Vera** and **Camomile** aroma. This unique formula is indicated for:

- Cleaning and daily hygiene of the nostrils to facilitate their functioning.
- Cleaning nasal passages when nasal congestion occurs.
- Moisturising the nostrils when nasal dryness is experienced.
- Cleaning the nostrils in allergic situations by helping to reduce the amount of allergens.
- Preventing the risk of complications such as otitis and sinusitis.

Posology

Repeat spraying as often as necessary to remove the mucus effectively.

respimar
DECONGESTANT

respimar
DECONGESTANT
GO

Indications

This is a patented formula with 3 exclusive and different herbal extracts for the relief of nasal congestion. The formula is designed for pathological processes accompanied by nasal secretion, with increased moisturisation due to the **Aloe Vera** and **Camomile** aroma. The **Mint** provides a pleasant sensation of fresh air and relief from the discomfort associated with nasal blockage due to excess mucus. This exclusive formula is indicated for:


- Decongestion of the nose for pathological processes that occur with increased mucus (e.g. cold, rhinitis, sinusitis or allergic rhinitis). The herbal extracts provide a moisturising and pleasant feeling.
- Prevention of the risk of complications, such as otitis and sinusitis.
- Decrease in the amount of allergens.
- Nasal cleaning and hygiene when nasal secretions are viscous and/or purulent.
- Can be used under medical advice as a supplement in the post-operative treatment of nasal surgery.
- Contribution to treatment for discomfort of the sinuses (sinusitis).

Posology

Respimar can be used as often and for as long as necessary. The recommended frequency is 1-3 sprays in each nostril 3-4 times a day while the excess mucus persists.

How to use respimar?

- 1 Blow your nose to remove mucus before using.
- 2 Remove the protective cap.
- 3 Tilt your head to one side over a sink and gently insert the spray into a nostril.
 For babies, lay the baby face up, turn its head to one side and gently insert the spray into a nostril.
- 4 Exert slight pressure on the spray to emit the sea water for **Respimar Pediatric**, **Respimar Cleaning and Hydration** and **Respimar Decongestant**.
For **Respimar Decongestant Go**, press the pump for the dosage of sea water. If it is the first time you have used **Respimar Decongestant Go** or you have not used it for a long time, you will need to charge the sprayer. This is done by pointing the container away from the body to avoid getting wet and pressing the spray several times until the liquid is expelled.
- 5 Let the solution act for a few seconds to remove the mucus and blow your nose.
- 6 Repeat the process with the other nostril after turning the head to the other side.
- 7 Clean the spray applicator with water and dry it.

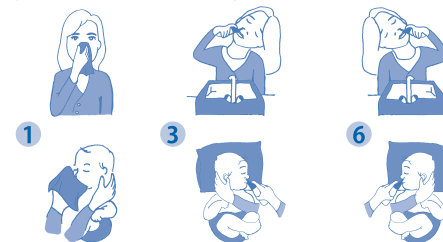Contraindications and side effects

- No adverse effects have been reported to the recommended therapeutic doses. If you experience any adverse effects, please contact your doctor or pharmacist.
- Any adverse incidents related to the product should be reported to the manufacturer, distributor or the competent Authority.

Warnings

- Keep out of the reach of children.
- Do not use the same applicator among several people.
- Store below 30°C.

Container

Respimar has two formats:

- **120 ml:** An innovative pack that does not need propellant gas or chemical preservatives. Thus, the sea water is preserved as 100% natural.
- **30 ml:** Adds portability and ease of use outside the home.

Respimar can be used for up to 12 weeks after opening: 120 mL pack
Respimar can be used for up to 4 weeks after opening: 30 mL pack

MD
Medical Device

CE
0051

STERILE R : 120ml packs

PR031794E17-23002

PHARMALINK, S.L.
Av. Universitat Autònoma, 13
08290 Cerdanyola del Vallès
Barcelona (Spain)
www.pharmalink.es
Customer service:
+34 900 102 796

For
Laboratorios Cinfa, S.A.
Olaz-Chipi, 10 P.I. Areta
31620 Huarte (Navarre)
Spain

www.cinfa.com

Date of issue: 13/04/2023

REF PL0078_120
REF PL0078_30
REF PL0079_AD
REF PL0079_BB

Knowing your
experience
with the
product will
help us to
grow

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